
F&P Nova Nasal

Instructions for Use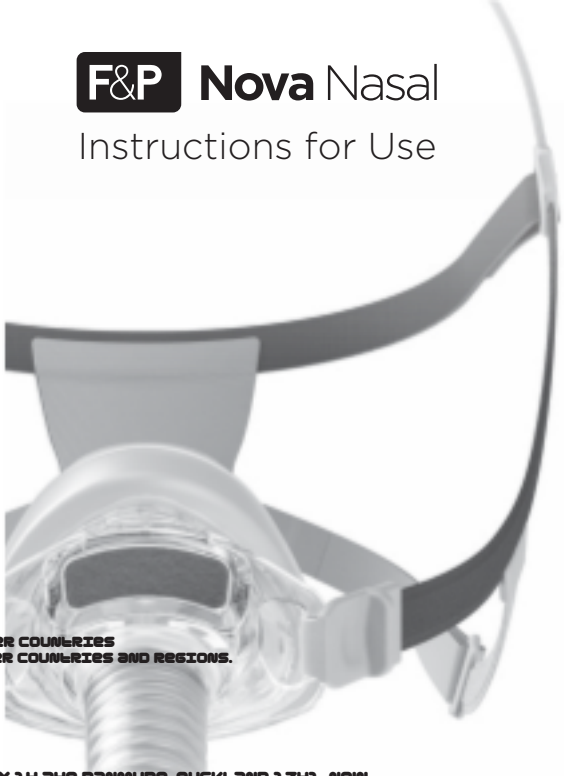

CE 0123

FSC LOGO HERE

GOOGLE PLAY IS A TRADEMARK OF GOOGLE LLC.
APPLE AND THE APPLE LOGO ARE TRADEMARKS OF APPLE INC., REGISTERED IN THE U.S. AND OTHER COUNTRIES
AND REGIONS. APP STORE IS A SERVICE MARK OF APPLE INC., REGISTERED IN THE U.S. AND OTHER COUNTRIES AND REGIONS.
REF 622488 REV A 2024.10 © 2024 FISHER & PAYKEL HEALTHCARE LIMITED

FISHER & PAYKEL HEALTHCARE LTD, 15 MAURICE PAYKEL PLACE, 6351 TAMAKI, AUCKLAND 2013, PO BOX 14 348 PARNMURE, AUCKLAND 1741, NEW ZEALAND TEL: +64 9 574 0100 EMAIL: INFO@FPHCARE.CO.NZ WEB: WWW.FPHCARE.COM

AUSTRALIA (AU) (SPONSOR) FISHER & PAYKEL HEALTHCARE PTY LTD, 19-31 KING STREET, MUNAWADING, MELBOURNE, VICTORIA 3131.

BRAZIL (BR) FISHER & PAYKEL DO BRASIL, RUA SAMPÃO VIANA, 277 CJ 21, PARAIISO, 04064-000, SÃO PAULO - SP, BRAZIL TEL: +55 11 2548 7002 CHINA (CN) 代理人/售后服务机构:费雪派克医疗保健（广州）有限公司, 广州高新

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(GB) FISHER & PAYKEL HEALTHCARE LTD, UNIT 16, CORDWALLIS PARK, CLIVEMONT ROAD, MAIDENHEAD, BERKSHIRE SL6 7BU, UK

TEL: 0000 132 169 USA (US)/CANADA (CA) TEL: 1800 446 3900 OR +1 949 453 4000

AUSTRIA (AT) TEL: 0000 29 31 23 BENELUX (BE NL LU) TEL: +31 40 216 3555 DENMARK (DK) TEL: +45 70 26 97 70 FINLAND (FI)

TEL: +358 9 251 66 123 IRELAND (IE) TEL: 1800 409 011 ITALY (IT) TEL: +39 06 45 25 70 95 NORWAY (NO) TEL: +47 21 60 13 53

SPAIN (ES) TEL: +34 902 013 346


Nasal Mask

FOR THE FULL SETUP EXPERIENCE,
DOWNLOAD THE F&P MYMASK™ APP*

















GET IT ON
Google Play

Download on the
App Store

* Not available in all countries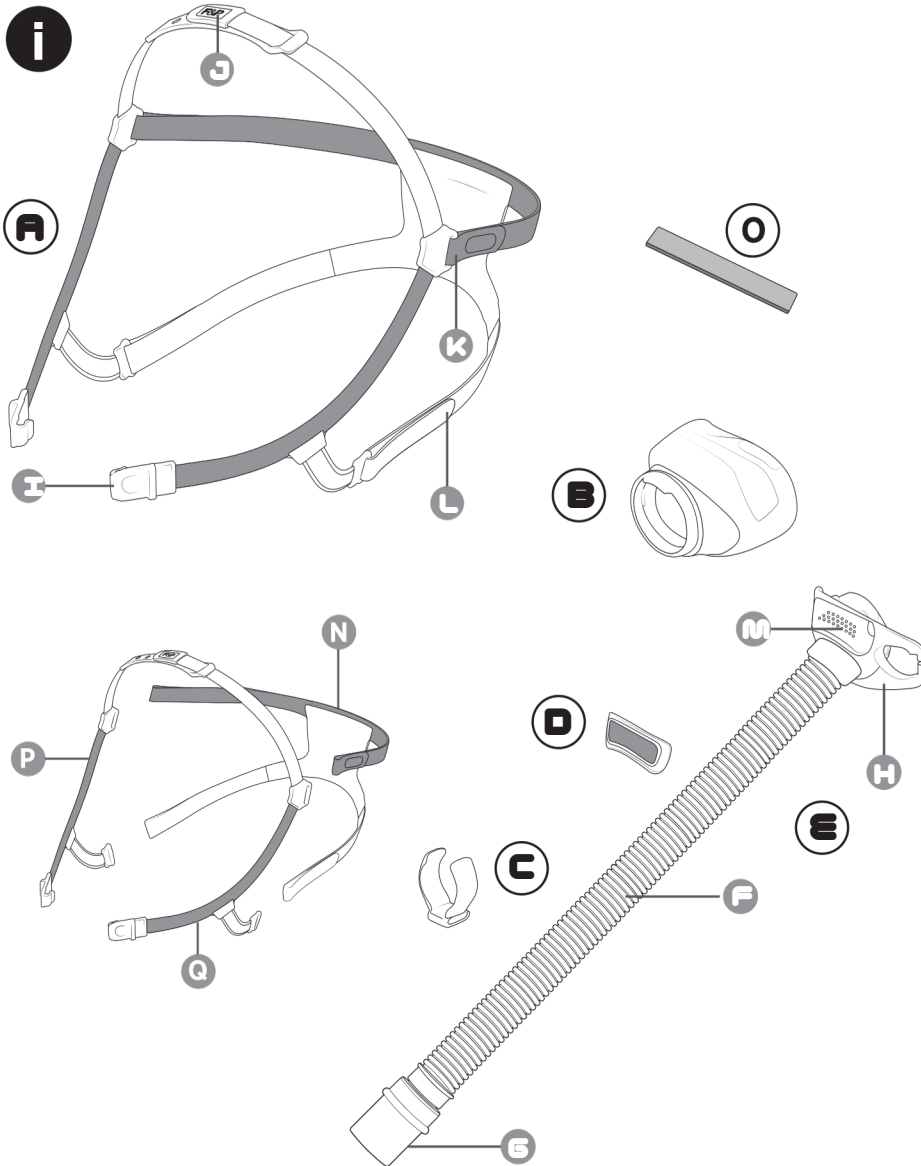

Fisher & Paykel
HEALTHCARE

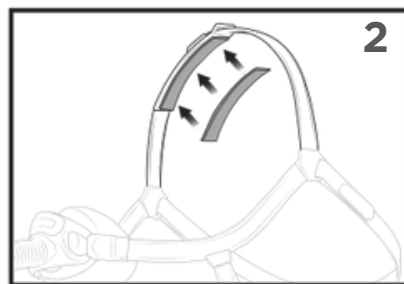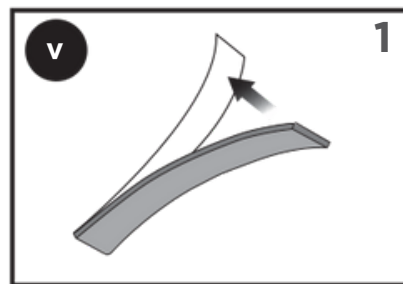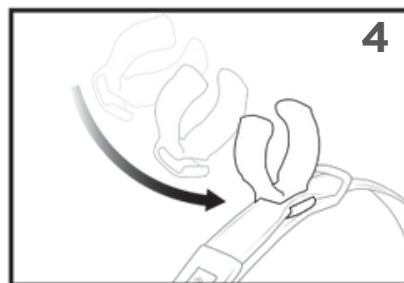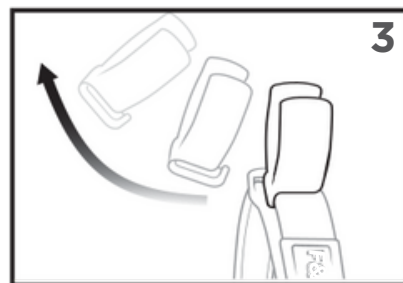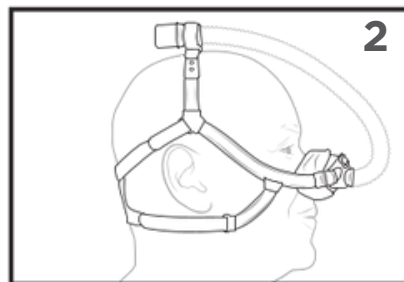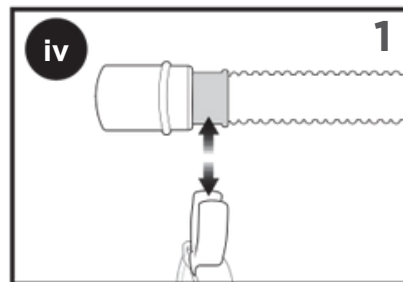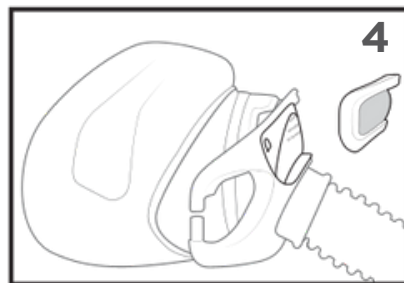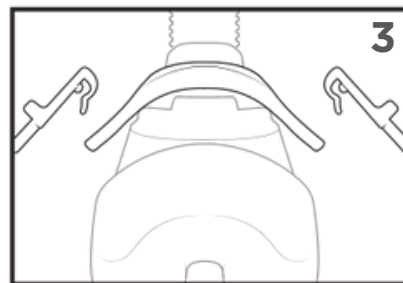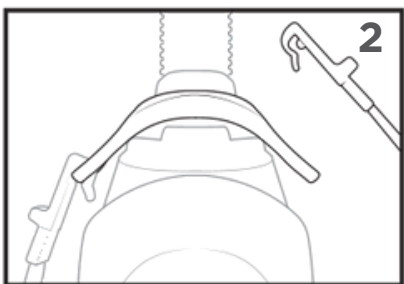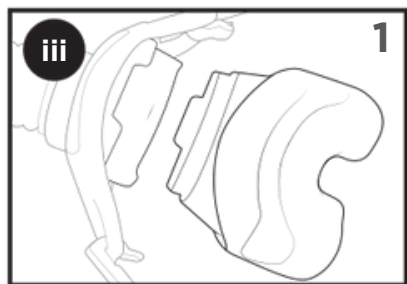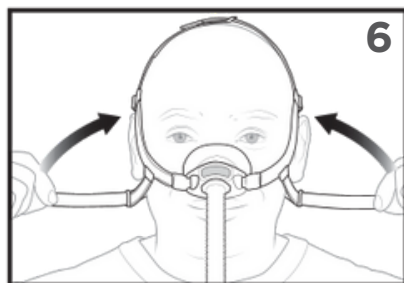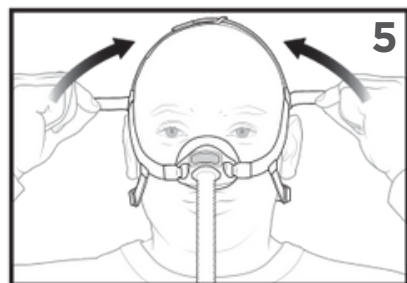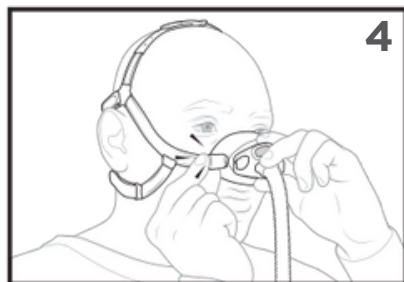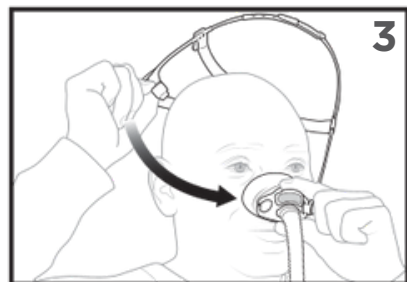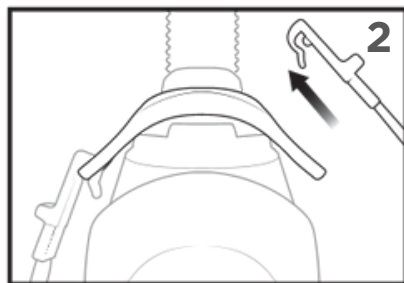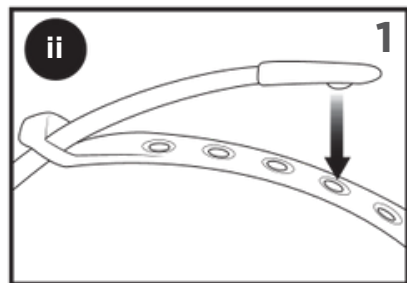PAGE LANGUAGE

6	ENGLISH
10	ESPAÑOL (SPANISH)
14	FRANÇAIS (FRENCH)
18	FRANÇAIS CANADIEN (CANADIAN)
22	DEUTSCH (GERMAN)
26	PORTUGUÊS (PORTUGUESE)
30	DANSK (DANISH)
34	NEDERLANDS (DUTCH)
38	SUOMI (FINNISH)
42	Ελληνικά (GREEK)
46	ITALIANO (ITALIAN)
50	NORSK (NORWEGIAN)
54	POLSKI (POLISH)
58	ROMÂNĂ (ROMANIAN)
62	SVENSK (SWEDISH)

Intended purpose

THE T&P NOVA™ NASAL MASK IS INTENDED TO BE USED IN INTERFACE IN A NON-INVASIVE POSITIVE AIRWAY PRESSURE (PAP) THERAPY SYSTEM TO DELIVER PRESSURIZED AIR FROM THE BREATHING TUBE TO THE PATIENT'S UPPER AIRWAY. THE PATIENTS ARE INTENDED TO BE ADULTS WEIGHING >55 LB (25 KG). THE MASK IS INTENDED TO BE USED FOR SINGLE PATIENT RE-USE IN THE HOME FOR THE TREATMENT OF A RESPIRATORY CONDITION AND IN A HOSPITAL OR OTHER CLINICAL SETTING TO INVESTIGATE THE EFFECT OF POSITIVE PRESSURE REQUIRED FOR THE TREATMENT OF A RESPIRATORY CONDITION.

General warnings and cautions

**ANY SERIOUS INCIDENT MUST BE REPORTED TO THE
AND YOUR FISHER & PAYKEL HEALTHCARE REPRESENTATIVE**

- WARNING: DO NOT USE THE MASK UNLESS THE PAP DEVICE IS TURNED ON AND OPERATING PROPERLY AS THIS MAY RESULT IN EXCESSIVE CARBON DIOXIDE INHALATION.**
- WARNING: THIS MASK IS NOT SUITABLE FOR LIFE SUPPORT VENTILATION AS THIS MAY RESULT IN DEATH.**
- WARNING: DO NOT ADMINISTER DRUGS OR MEDICATIONS INTO THE DEVICE UNLESS IT IS THE EXCEPTION OF SO-CALLED "EMERGENCY DRUGS" AS IT MAY IMPACT THE EFFICACY OF THE MEDICINE.**
- WARNING: DO NOT USE THE MASK OUTSIDE OF THE INTENDED PURPOSE AS THIS MAY RESULT IN EXCESSIVE LEAKS BETWEEN THE CUSHION AND THE MASK RESULTING IN COMPROMISED THERAPY. IT CAN CAUSE RESPIRATORY DEPRESSION USING THE MASK WITHOUT PROPER RESCUE.**

Possible side effects:

WHILE USING THE F&P NOVA NASAL MASK, YOU MAY EXPERIENCE DRY EYES OR DRY NOSE DURING SLEEPING.

1. DISCONTINUE USE OF THE F&P IMMEDIATELY IF YOU EXPERIENCE ANY OF THE FOLLOWING SIDE EFFECTS:
 • ALLERGY REACTION (REDNESS, SWELLING, BURNING, ITCHING, OR OTHER REACTION) PROVIDER OR PHYSICIAN IF YOU
 • ORAL PATCH REACTION (REDNESS AND/OR BURNING/ITCHING), OR
 • DERMATITIS (REDNESS AND/OR BURNING/ITCHING) AND/OR
 • CESSATION OF BREAST AND/OR ANXIETY.
 • EYE DRYNESS, SWELLING AND/OR REDNESS. IF YOU
 EXPERIENCE ANY OF THE ABOVE SIDE EFFECTS,
 INCLUDING ANY NOT LISTED IN THESE
 INSTRUCTIONS, DISCONTINUE USING THE F&P
 NOVA NASAL MASK AND CONSULT YOUR PHYSICIAN
 IMMEDIATELY. ANY OTHER SERIOUS INCIDENTS
 MUST BE REPORTED TO THE COMPETENT
 AUTHORITY AND YOUR FISHER & PAYKEL
 HEALTHCARE REPRESENTATIVE.

REFER TO THE MASK DIAGRAM IN THESE INSTRUCTIONS.

- (A) HEADGEAR (B) CUSHION (C) TUBE CLIP
(D) DIFFUSER (E) FRAME ASSEMBLY (F) TUBE
(G) SWIVEL (H) FRAME (I) HEADGEAR CLIPS
(J) TOP STRAP (K) UPPER STRAPS (L) LOWER STRAPS
(M) EXHAUST HOUSING LEFT PANEL (N) HEADGEAR P30
(P) RIGHT SIDEARM (Q) LEFT SIDEARM

Operating instructions

PLEASE REFER TO THE WARNINGS AND CAUTIONS FOLLOWING THESE INSTRUCTIONS. FAILURE TO FOLLOW THE OPERATING INSTRUCTIONS MAY COMPROMISE THE PERFORMANCE AND SAFETY OF THE MASK. ONLY USE YOUR MASK FOR ITS INTENDED PURPOSE AS DIRECTED IN THESE INSTRUCTIONS. PRESSURE RANGE OF YOUR MASK IS 4 TO 30

- WARNING: ONLY USE THE MASK WITHIN THE SPECIFIED PRESSURE RANGE.**
- USE OUTSIDE OF THE RANGE MAY RESULT IN INHALATION OR EXCESSIVE LEAK WHICH MAY RESULT IN AN EYE INJURY. WARNING: YOUR MASK SHOULD ONLY BE USED WITH APPROVED TUBING AND A PRESSURE RANGE OF YOUR MASK IS 5 CM H₂O OR RESPIRATORY**
- STORAGE: DO NOT USE THE MASK WITH ANY OTHER MEDICAL DEVICES COULD.**
- PERFORMANCE OF THE MASK MUST BE IN DIRECT SUNLIGHT COMPROMISE THE FUNCTION OF THE MASK.**
- THE EXPIRY DATE IS PRINTED ON THE LABEL OF THE MASK.**
- YOUR MASK AFTER ITS EXPIRY DATE.**
- FOR SINGLE PATIENT USE IN THE HOME, REPLACE THE MASK 12 MONTHS AFTER THE DATE OF FIRST USE.**

Before use

BEFORE USING YOUR MASK THE FIRST TIME, REMOVE ALL
BEFORE USING YOUR MASK EACH TIME:

- WARNING:** MAKE SURE THE AIR PATH THROUGH THE MASK BLOCKAGES THAT MAY PREVENT OR RESTRICT AIRFLOW IN EXCESSIVE CARBON DIOXIDE INHALATION.
- WARNING:** INSPECT THE EXHAUST HOLES (M). ENSURE AIR THROUGH THE EXHAUST AND DO NOT USE IF BLOCKED AS YOUR FACE AND CARBON DIOXIDE INHALATION.
- CAUTION:** INSPECT IT FOR DAMAGE OR DETERIORATION AND SIGNIFICANT DEGRADATION (CRACKING, TEARS, ETC.). DO AND SEEK REPLACEMENT PART(S) FROM YOUR HEALTHCARE AS USING THE MASK MAY RESULT IN LACERATIONS OR THERAPY.
- PERMISSION:** INSPECT THE MASK FOR SOILING AND DUST. FOLLOW THE CLEANING INSTRUCTIONS TO REMOVE THE SOILING. THIS MAY RESULT IN A RANGE OF HARMS, SOME OF WHICH

Fitting your mask (ii)

- REFER TO THE FITTING DIAGRAMS IN THESE INSTRUCTIONS TO CORRECTLY ORIENT THE HEADGEAR (A) ON THE FRAME (H).
1. CONNECT THE TOP STRAP (J) TO ONE OF THE HOLES IN THE HEADGEAR (A) AND THE HEADGEAR (A) TO THE FRAME (H).
 2. UNHOOK ONE OF THE HEADGEAR CLIPS (I) FROM THE FRAME (H) AND THE DIFFUSER (D) SHOULD BE REMOVED.
 3. PLACE THE CUSHION (B) ON YOUR NOSE AND SWING THE HEADGEAR CLIP (I) AROUND YOUR HEAD.
 4. CONNECT THE HEADGEAR CLIP (I) TO THE FRAME (H) AND THE HEADGEAR (A) TO THE FRAME (H). YOU SHOULD HEAR A CLICK WHEN CONNECTED PROPERLY.
 5. GENTLY ADJUST THE UPPER STRAP (K).
 6. GENTLY ADJUST THE LOWER STRAP (L).

USEFUL TIPS:

- THE HEADGEAR (A) SHOULD NOT TOUCH YOUR EARS. IF IT DOES, YOU MAY NEED TO ADJUST THE UPPER STRAP (K) IF REQUIRED. YOU MAY NEED TO TAKE CARE OF THE HEADGEAR (A) AND THE HEADGEAR CLIP (I) IF REQUIRED.
- IF YOU ARE EXPERIENCING MASK LEAKS, YOU MAY NEED TO ADJUST THE CUSHION (B) OFF YOUR FACE AND REPOSITION IT. THE SLICKY ADHESIVE LAYER.
- FOR A COMFORTABLE FIT, THE MASK SHOULD BE WORN ON THE ADHESIVE SIDE OF THE HEADGEAR (A) POSSIBLE, WITHOUT EXCESSIVE LEAK.

Using optional Headgear Pad (iii)

1. REMOVE THE HEADGEAR PAD (N) FROM THE HEADGEAR (A) AND THE HEADGEAR CLIP (I) FROM THE FRAME (H). REMOVED ONCE IT HAS BEEN APPLIED. TO USE THE HEADGEAR PAD (N), YOU MUST REMOVE THE SLICKY ADHESIVE LAYER FROM THE HEADGEAR (A) AND THE HEADGEAR CLIP (I) FROM THE FRAME (H). FIRMLY TO SECURE IT TO THE RIGHT SIDEARM.

Cleaning your mask at home

- WARNING: DO NOT OVERTIGHTEN THE MASK AS THIS MAY RESULT IN FACIAL DEFORMATION.**
- CAUTION: FAILURE TO FOLLOW FITTING INSTRUCTIONS OR USING THE MASK AFTER EACH USE: WASH THE CUSHION (B) AND THE HEADGEAR (A) IN SOAP DISSOLVED IN WATER. WASH THE MASK AFTER 7 DAYS USE: WASH THE MASK (D).**

Taking on your mask

1. UNHOOK ONE OF THE HEADGEAR CLIPS (I) FROM THE FRAME (H) AND THE DIFFUSER (D) SHOULD BE REMOVED.
2. UNWRAP THE HEADGEAR (A) FROM YOUR HEAD. THE UPPER STRAP (K), LOWER STRAP (L) AND THE TOP STRAP (J) DO NOT HAVE TO BE REMOVED FOR MORE THAN 10 MINUTES.
3. RINSE THOROUGHLY IN FRESH WATER. ENSURE THAT ALL BEEN REMOVED.
4. INSPECT ALL PARTS TO ENSURE THEY ARE VISIBLY CLEAN.

Disassembling your mask (iii)

- REFER TO THE DISASSEMBLY DIAGRAMS IN THESE INSTRUCTIONS TO CORRECTLY ORIENT THE HEADGEAR (A) ON THE FRAME (H).
1. REMOVE THE CUSHION (B) BY PULLING IT AWAY FROM THE FRAME (H).
 2. REMOVE THE HEADGEAR (A) BY PUSHING ONE OF THE HEADGEAR CLIPS (I) FORWARD UNTIL IT DISCONNECTS FROM THE FRAME (H).
 3. REMOVE THE OTHER HEADGEAR CLIP (I) BY REPEATING STEP 2.
 4. TO REMOVE THE DIFFUSER (D), USE YOUR FINGER TO LOOSEN THE DIFFUSER (D) FROM THE FRAME (H).

- CAUTION: THE FRAME (H), TUBE (F) AND SWIVEL (G) COMPONENTS SPECIFIED IN THESE INSTRUCTIONS ARE PERMANENTLY CONNECTED TOGETHER AND ARE NOT INTENDED TO BE DISASSEMBLED. ATTEMPTING TO DISASSEMBLE THESE COMPONENTS MAY RESULT IN AN ADVERSE REACTION IF CLEANING RESIDUE IS LEFT BEHIND.**

Assembling your mask (iv)

- REFER TO THE ASSEMBLY DIAGRAMS IN THESE INSTRUCTIONS TO CORRECTLY ORIENT THE HEADGEAR (A) ON THE FRAME (H).
1. ATTACH THE CUSHION (B) BY PUSHING IT ONTO THE FRAME (H). YOU SHOULD HEAR A CLICK WHEN CONNECTED PROPERLY.
 2. ATTACH THE HEADGEAR (A) BY ALIGNING BOTH HEADGEAR CLIPS (I) WITH THE FRAME (H) AND PULLING BACK UNTIL YOU HEAR A CLICK WHEN CONNECTED PROPERLY.
 3. ATTACH THE OTHER HEADGEAR CLIP (I) BY REPEATING STEP 2.
 4. IF YOU NEED TO REATTACH THE DIFFUSER (D), ALIGN THE DIFFUSER (D) WITH THE FRAME (H) AND PUSH TO CONNECT.

Using optional Tube Clip (v)

- REFER TO THE TUBE CLIP DIAGRAMS IN THESE INSTRUCTIONS TO CORRECTLY ORIENT THE HEADGEAR (A) ON THE FRAME (H).
1. FIND THE TUBE CLIP (C) ON THE TOP STRAP (J). THE TUBE CLIP (C) WILL HAVE TO BE FITTED BY PUSHING IT INTO THE TUBE CLIP (C) OR REMOVED BY PULLING IT OUT.
 2. IF YOU HAVE FITTED THE TUBE (F) TO THE TUBE CLIP (C) AND THE TUBE CLIP (C) IS NOW ON THE TOP OF YOUR HEAD.
 3. THE TUBE CLIP (C) CAN BE REMOVED BY ROTATING THE TUBE CLIP (C) FROM THE TOP STRAP (J).
 4. TO ATTACH THE TUBE CLIP (C), HOOK AND ROTATE IT INTO THE TOP STRAP (J).

Oxygen/pressure port connector

IF PRESSURE READINGS AND/OR ADDITIONAL OXYGEN ARE REQUIRED, AN F&P THE SOUND POWER LEVEL OF THE MASK WITH UNCERTAINTY 2.5 DBA. OXYGEN/PRESSURE PORT CONNECTOR IS AVAILABLE (REF 990HC452). PLEASE WITH UNCERTAINTY 2.5 DBA. CONTACT YOUR HEALTHCARE PROVIDER FOR MORE INFORMATION. PRESSURE LEVEL OF THE MASK WITH DIFFUSER WARNING: MASK USE WITH SUPPLEMENTAL OXYGEN WILL CREATE A RISK WITH UNCERTAINTY 2.5 DBA. OF FIRE. DO NOT SMOKE, OR BE IN THE VICINITY OF SOMEONE SMOKING, THE SOUND POWER LEVEL OF THE MASK OR ANY OTHER IGNITION SOURCE WHILE USING SUPPLEMENTAL OXYGEN WITH UNCERTAINTY 2.5 DBA. WITH YOUR MASK. THIS MAY RESULT IN IGNITION OF THE OXYGEN, LEADING TO THE SOUND PRESSURE LEVEL OF 60 SERIOUS BURNS OR DEATH. WITH UNCERTAINTY 2.5 DBA.

Use with other PAP devices


WARNING: WHEN USING THE F&P NOVA NASAL MASK WITH OTHER THE F&P NOVA NASAL MASK AND ACCESSORIES 22 MM CONNECTOR (ISO 5956-1) COMPATIBLE PAP DEVICES MADE WITH NATURAL RUBBER LATEX OR DEHP. (INCLUDING OXYGEN/PRESSURE PORT CONNECTOR) IF YOU ARE ALLERGIC TO ANY OF THE FOLLOWING MATERIALS, CONSULT YOUR PHYSICIAN BEFORE USING THE F&P NOVA NASAL MASK, AS THIS MAY CAUSE AN ADVERSE REACTION.

Technical specifications

F&P Nova Nasal mask exhaust flow
THE F&P NOVA NASAL MASK HAS A UNIQUE DIFFUSER SYSTEM THAT IS DESIGNED TO DIFFUSE AIR WHICH IS EXPELLED FROM THE MASK. THIS REDUCES DRAFT. THE DIFFUSER DOES NOT SIGNIFICANTLY CHANGE THE AMOUNT OF AIR FLOWING FROM THE MASK; THEREFORE, THE MASK CAN BE USED WITH OR WITHOUT THE DIFFUSER.

Exhaust flow information	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	22	24	26	28	30
PRESSURE (CMH ₂ O)	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	22	24	26	28	30
FLOW WITH DIFFUSER (L/MIN)	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	22	24	26	28	30
FLOW WITHOUT DIFFUSER (L/MIN)	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	22	24	26	28	30

DUUE TO MANUFACTURING VARIATIONS, THE EXHAUST FLOW MAY VARY FROM THE NOMINAL VALUES SHOWN ABOVE.

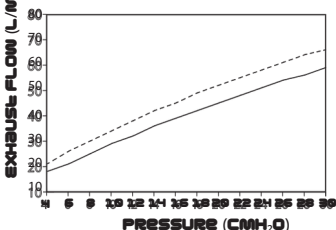Resistance to flow

PRESSURE DROP THROUGH THE MASK WITH DIFFUSER AT 50 L/MIN: 0.3:0.1 CMH₂O.
PRESSURE DROP THROUGH THE MASK WITHOUT DIFFUSER AT 50 L/MIN: 0.3:0.1 CMH₂O.
PRESSURE DROP THROUGH THE MASK WITH DIFFUSER AT 100 L/MIN: 1.5:0.3 CMH₂O.
PRESSURE DROP THROUGH THE MASK WITHOUT DIFFUSER AT 100 L/MIN: 1.5:0.3 CMH₂O.

Mask dead space

SMALL = 69.6 CM³ MEDIUM = 75.0 CM³ LARGE = 86.8 CM³

Sound

Mask materials

COMPONENT	ADVERSE REACTION
HEADGEAR	ACETAL (POM), NYLON (PA), POLYETHER BLOCK AMIDE (PEBA), POLYURETHANE (PU)
CUSHION	SILICONE, POLYCARBONATE (PC)
FRAME	POLYCARBONATE (PC)
DIFFUSER	POLYCARBONATE (PC), POLYETHYLENE TEREPHTHALATE (PET)
TUBE CLIP	ACETAL (POM)
TUBE	POLYCARBONATE (PC), POLYURETHANE (PU)
SEWEL	NYLON (PA)
HEADGEAR PAD	ACRYLIC (PMMA), POLYETHYLENE (PE)

Replacement parts

THE FOLLOWING PARTS ARE AVAILABLE FOR REPLACEMENT. HEALTHCARE PROVIDER FOR MORE INFORMATION.

PRODUCT CODE	DESCRIPTION	
4089VN11, 4089VN12, 4089VN13	F&P NOVA NASAL SPARE CUSHION	SMALL MEDIUM LARGE
4089VN11, 4089VN12, 4089VN13	F&P NOVA NASAL SPARE CUSHION	SMALL MEDIUM LARGE
4089VN11, 4089VN12, 4089VN13	F&P NOVA NASAL SPARE CUSHION	SMALL MEDIUM LARGE
4089VN11, 4089VN12, 4089VN13	F&P NOVA NASAL SPARE CUSHION	SMALL MEDIUM LARGE
4089VN12, 4089VN13	F&P NOVA NASAL BACK PANEL - SMALL	
4089VN12, 4089VN13	F&P NOVA NASAL BACK PANEL - MEDIUM	
4089VN12, 4089VN13	F&P NOVA NASAL BACK PANEL - LARGE	
4089VN131*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN132*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN133*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN134*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN135*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN136*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN137*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN138*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN139*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN140*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN141*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN142*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN143*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN144*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN145*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN146*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN147*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN148*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN149*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN150*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN151*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN152*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN153*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN154*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN155*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN156*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN157*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN158*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN159*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN160*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN161*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN162*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN163*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN164*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN165*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN166*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN167*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN168*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN169*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN170*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN171*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN172*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN173*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN174*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN175*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN176*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN177*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN178*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN179*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN180*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN181*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN182*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN183*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN184*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN185*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN186*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN187*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN188*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN189*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN190*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN191*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN192*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN193*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN194*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN195*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN196*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN197*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN198*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN199*	F&P NOVA NASAL MASK - MASK NO 4089VN13	
4089VN200*	F&P NOVA NASAL MASK - MASK NO 4089VN13	

Warranty statement

FISHER & PAYKEL HEALTHCARE WARRANTS THAT THE MASK (EXCLUDING FOAMS AND MATERIAL DIFFUSERS), WHEN USED IN ACCORDANCE WITH ITS INSTRUCTIONS FOR USE, SHALL BE FREE FROM DEFECTS IN WORKMANSHIP AND MATERIALS AND WILL PERFORM IN ACCORDANCE WITH FISHER & PAYKEL HEALTHCARE'S OFFICIAL PUBLISHED PRODUCT SPECIFICATIONS FOR A PERIOD OF 90 DAYS FROM THE DATE OF PURCHASE BY THE END USER. THIS WARRANTY IS SUBJECT TO THE FOLLOWING: THIS WARRANTY DOES NOT COVER HAZARDOUS MATERIALS AND IS NOT RESPONSIBLE FOR GENERAL INJURY, Intellectual property information

F&P, F&PNOV2220 AND MYMASK ARE TRADEMARKS OF FISHER & PAYKEL HEALTHCARE LIMITED. FOR PATENT INFORMATION, SEE WWW.FPHCARE.COM/IP

Symbol definitions

	CATALOGUE NUMBER		CE MARKING
Rx only	PRESCRIPTION ONLY		AUTHORIZED EUROPEAN REPRESENTATIVE
	SMALL CUSHION SIZE		MEDIUM CUSHION SIZE
	LARGE CUSHION SIZE		HUMIDITY LIMITATION
	USE BY DATE		STORAGE TEMPERATURE LIMITATION
	DATE OF MANUFACTURE		CONSULT INSTRUCTIONS FOR USE
	UNIQUE DEVICE IDENTIFIER		BLUE SYMBOL: FOLLOW INSTRUCTIONS FOR USE
	MEDICAL DEVICE		CAUTION/WARNING
	BATCH CODE		MANUFACTURER
	RECYCLING SYMBOL		SMALL HEADGEAR SIZE
	MEDIUM-LARGE HEADGEAR SIZE		SINGLE PATIENT MULTIPLE USE
	LOW-DENSITY POLYETHYLENE		POLYETHYLENE TEREPHTHALATE, LOW-DENSITY POLYETHYLENE
	IMPORTER		DISTRIBUTOR
	SWITZERLAND AUTHORIZED REPRESENTATIVE		UK RESPONSIBLE PERSON

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