





Mirage Micro™
 NASAL MASK
Mirage Activa™ LT
 NASAL MASK
Mirage™ SoftGel
 NASAL MASK

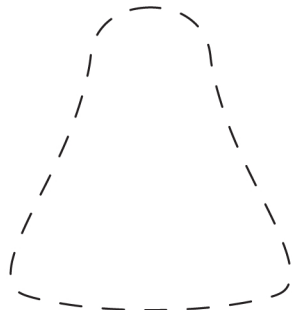


↑
SMALL

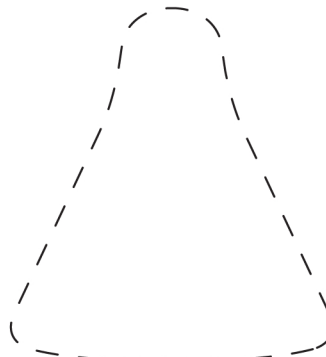


↑
MEDIUM

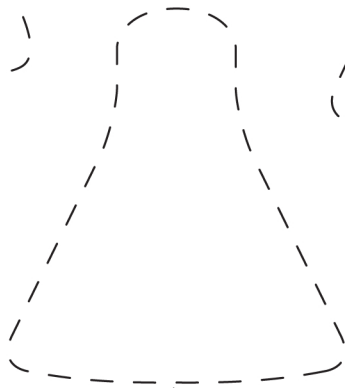

↑
LARGE

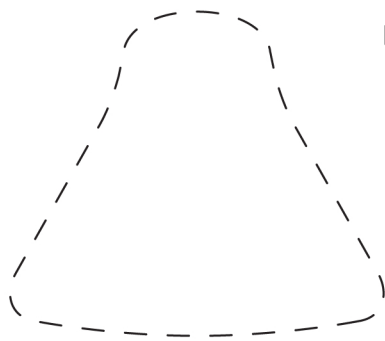

↑
LARGE WIDE

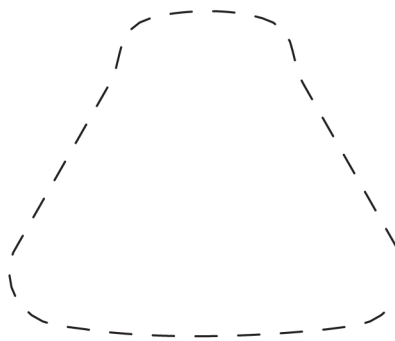

↑
X-LARGE
(Micro only)



How to use the fitting template:

1. Cut out the template along the dotted line
2. Cut out nasal holes
(hint: fold each nasal hole in half to cut)
3. Follow directions on template to determine appropriate cushion size

Actual Size: 5½" (Width) x 7" (Height)

Ruler is for scaling purposes.

RESMED